



medical brain®

# REMOTE PATIENT MONITORING (RPM) PROGRAM

## **PULSE OXIMETER** QUICK START GUIDE

Medical Brain has partnered with Wellue® to bring you a best-in-class self monitoring fingertip oximeter device which meets U.S. Food & Drug Administration (FDA) guidelines.

You will find in this guide:

- All steps to connect this device to the Medical Brain app by taking your first reading
- About the Medical Brain and its app features



medical brain®

## **WELCOME,**

Your provider has selected you for the Medical Brain Remote Patient Monitoring (RPM) Program with YOUR healthcare needs in mind. We're linked to your provider. No gaps, no delays.

**We're here for your health!**

The Medical Brain Team

## **The Medical Brain RPM Program provides you with:**



Your pulse oximeter selected for you by your provider



Your Medical Brain app – your personal health assistant that always understands your health needs and takes immediate actions to prevent problems

Should you experience a change in your health, the Medical Brain will notify your provider for early intervention and treatment



Your dedicated Medical Brain Care Team, in partnership with your provider, giving healthcare support when you need it the most

# 1

Get started


## Open the Medical Brain app

- Tap on the “Chat” icon
- Type or use the Microphone icon to tell the Medical Brain “I am ready”

Medical Brain will respond to you with a “Start” button

- Tap on the button to start
- Follow steps

Note: This is a one-time process needed to connect your device to the app

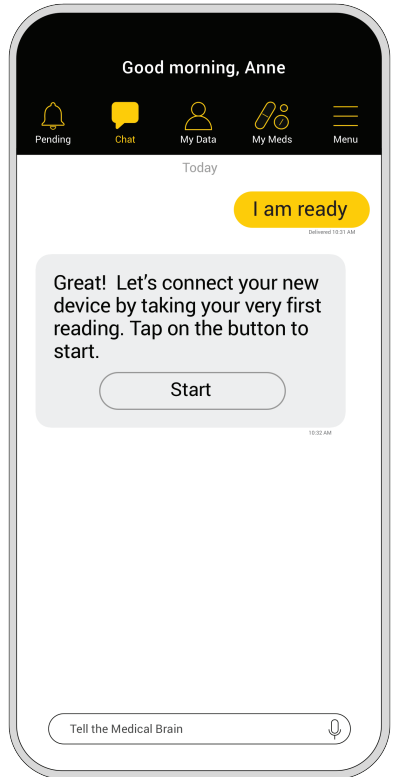


## 2

### Get your phone ready to connect with your new device


Ensure Bluetooth is turned ON in your phone

If OFF, how do I turn it ON?

- 1 Go to your phone's home screen
- 2 Tap on the "Settings" icon
- 3 Look for  "Bluetooth" in the list
- 4 Tap the "Bluetooth" switch to turn it ON

# 3

## Install the included batteries into your pulse oximeter

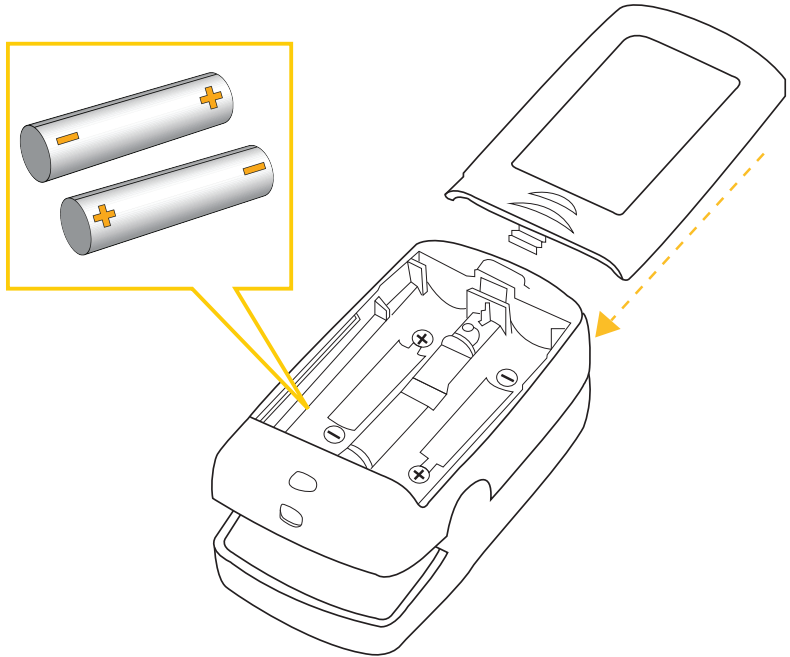

Install batteries with the positive (+) and negative (-) terminals properly aligned with the corresponding symbols in the battery compartment


# 4

## Take first reading

Open the clip and put finger inside the rubber cushions of the clip (make sure the finger is in the correct position), and then clip the finger

Wait 2 seconds, the Oximeter will power on automatically and start to measure

Your measurement will be sent to the Medical Brain

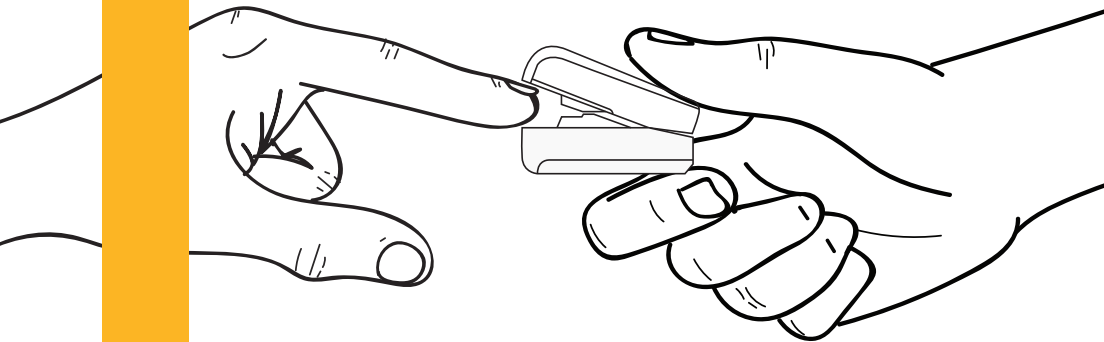



# 5

## Allow permissions

At this point, your pulse oximeter is connecting with your phone and the Medical Brain app

To enable connection, please allow all requests for Bluetooth and/or Location

Not allowing these permissions will stop the pairing (connecting process)

**"Medical Brain"**  
Would like to Use Bluetooth

It allows you to connect Bluetooth devices and automatically import your data into Medical Brain

DON'T ALLOW

OK

**Allow "Medical Brain"**  
to access this device's location?


DENY

ALLOW

# 6

## View measurement

- Medical Brain will present your first measurement
- Tap on “Done” to see it added to your chat

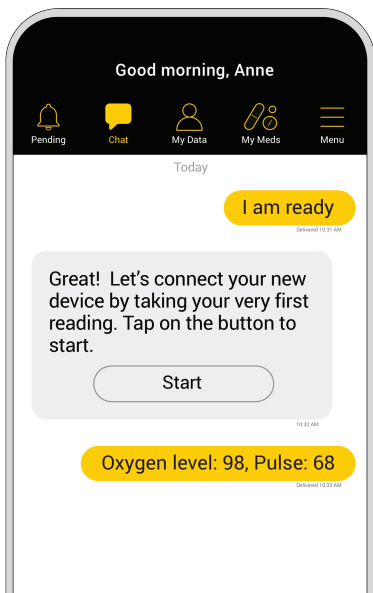

### Congratulations

Your first measurement is  
Oxygen level: 98, Pulse: 68

### For future measurements:

Simply open the app, use your  
device and confirm your  
measurement when prompted

DONE

Note: If you received more than one device, you will be redirected to “Measurements” screen to set up the next device




## Track your measurements

All your measurements will be saved and displayed in the Medical Brain app under "My data">"Measurements">"Oxygen level">"History"

### How to take readings regularly?

Just open the app, use your oximeter and confirm your measurement when prompted

Measurement will be shown in "Chat" and logged under "History" for you and your provider to see

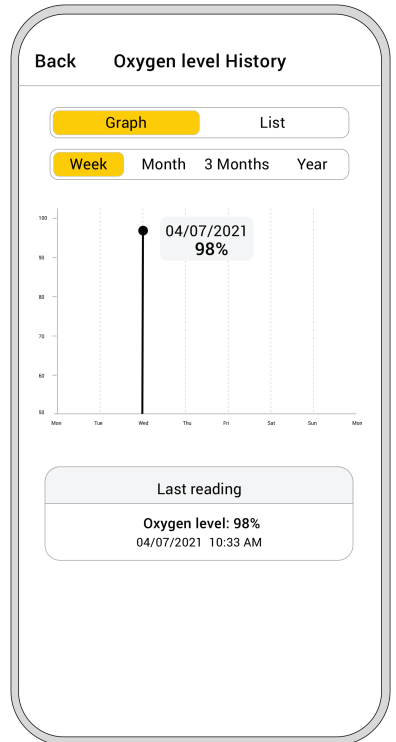



medical brain®

## What is the Medical Brain?

Medical Brain is your intelligent personal health assistant that collaborates with your healthcare provider and helps you manage certain diseases, such as cardiovascular, hypertension, diabetes, COPD, Covid-19 and more


### What does the Medical Brain do?



Continuously analyzes your health status, based on your medical background and communications by you and your provider with the Medical Brain



# Notifies your care team for early intervention and treatment should you experience a change in your health

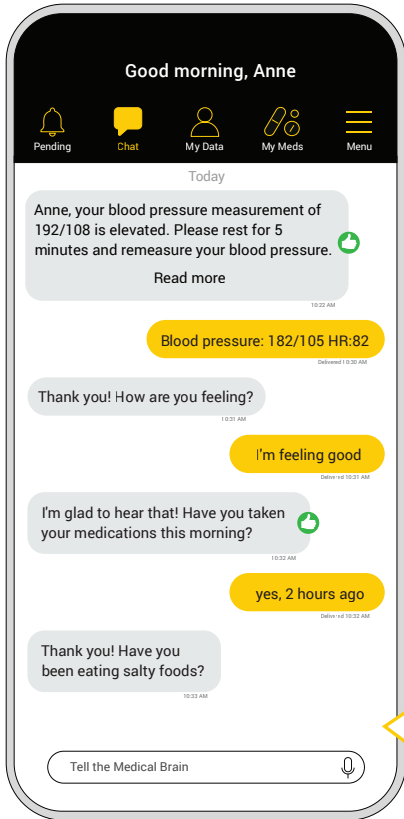

No. I've been good about limiting my salt.

Delivered 10:24 AM

Thank you for letting me know. Your blood pressure of 182/105 is elevated. Provider evaluation is indicated.

10:25 AM

Dr. Smith has been notified. Your blood pressure medications may need to be adjusted.

10:26 AM

Dr. Smith wants you to take an extra dose of Amlodipine 5mg now and to increase your Amlodipine to 10mg daily starting tomorrow.

Please remeasure your blood pressure in 2 hours to make sure your blood pressure is responding to the medication.

If you develop any symptoms such as severe headache, chest pain, shortness of breath, please go to the Emergency department for evaluation.

10:40 AM

Thank you! will do. I'm taking my amlodipine now

Delivered 10:47 AM



Generates an Intelligent Medical Summary (IMS) of all important information about your health history and current health status



Connects you and your care team 24/7: Assists you in scheduling appointments, communicating with your provider, and keeping your provider up-to-date with your health status, all through the Medical Brain's intelligent chat

My Data

Measurements IMS Lab Results Test Reports

INTELLIGENT MEDICAL SUMMARY

ACUTE PROBLEMS		
Hypertensive urgency		03/18/2021

SIGNIFICANT VITALS

Blood Pressure	182/105	03/18/2021 10:30
Heart Rate	82	03/18/2021 10:30

RELEVANT LAB AND DIAGNOSTIC RESULTS

Creatinine	0.89	02/10/2021
eGFR	65	02/10/2021
Potassium	4.4	02/10/2021
Ejection Fraction	62%	03/15/2020

CURRENT MEDICAL DIAGNOSIS

Hypertension	Exist	06/10/2015
Hypercholesterolemia	Exist	06/10/2015
Coronary Artery Disease	Exist	01/23/2017



Immediately displays labs and test reports when available for you and your provider to view



Speak (or text) with the Medical Brain anytime. The Medical Brain understands you and knows your health profile for accuracy and care coordination



Keeps all your communications and data safe and secured

The image shows a smartphone screen with a dark theme. At the top, there is a back arrow and the title 'My Data'. Below the title are four icons: 'Measurements' (two overlapping squares), 'IMS' (a document with a checkmark), 'Lab Results' (a flask), and 'Test Reports' (a document with a plus sign). The main content area is titled 'LAB RESULTS' and shows a table of test results. The table has three columns: 'Test', 'Results', and 'Normal Range'. The data is as follows:

Test	Results	Normal Range
CBC/DIFF - 02/10/2021, 4:28 AM		
WBC COUNT	7.7	4.8-10.8
RBC COUNT	5.25	4.70-6.10
HEMOGLOBIN	15.3	14.0-18.0
HEMATOCRIT	47.1	42.0-52.0
MCV	89.7	80.0-94.0
MCH	29.1	27.0-31.0
MCHC	32.5	33.0-37.0
RDW	13.0	11.4-16.4
PLATELET COUNT	226	150-450
DIFF TYPE	AUTOMATED	
NEUTROPHILS-5P	84.3	37.9-70.5
LYMPHOCYTES-5P	6.7	19.8-47.7
MONOCYTES-5P	7.7	2.7-11.7
EOSINOPHILS-5P	0.3	0.3-5.9
BASOPHILS-5P	0.1	0.1-2.4
IMM GRANULOCYTE %	0.9	
ABS NEUTROPHILS	6.5	1.4-6.5

At the bottom right of the screen, there is a circular icon with an upward-pointing arrow, likely for sharing or printing the data.



medical brain®

**We're all about your health  
and we're all about you.**

For customer service and any technical questions  
about your devices, please call (888) 355 0269